Digital Bathroom Scale

Model HD-380WW/382WW

en Instruction Manual

Read this Instruction Manual carefully and keep it for future reference. Images in this manual are for illustrations purposes only.

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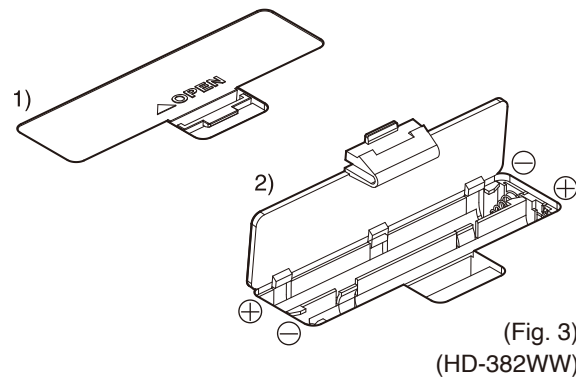
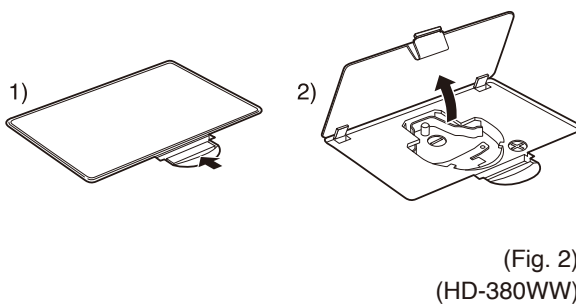
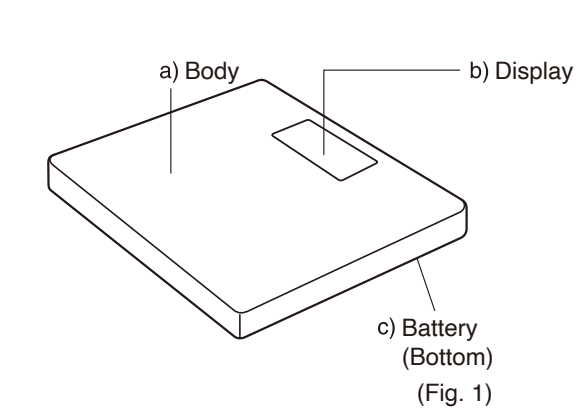
en INSTRUCTIONS FOR USE

For Your Safety

This section explains precautionary measures to be taken to avoid injury to the users of this device and others, and to prevent damage to property. Please familiarize yourself with this information to ensure safe operation of this equipment.

	Warning	Failure to follow instructions highlighted with this mark could result in death or severe injury.
	Caution	Failure to follow instructions highlighted with this mark could result in injury or damage to property.
		This mark indicates actions that are prohibited.
		This mark indicates instructions that must always be followed.

Warning

	Do not use the product if there are any signs of damage (cracks, etc.) or missing parts on the exterior before or during use.
	Do not heat this unit or batteries. There is a risk of injury or fire due to the deformation of this unit or the ignition of the battery.
	Battery must be kept out of reach of infants. It is possible to be swallowed by mistake. In case of accidental swallowing, please seek medical attention immediately.
	Do not step on the edge of the platform. Do not jump onto, or on the monitor. Doing so may cause you to fall or slip, resulting in serious injury.
	Do not place this monitor on a slippery surface such as a wet floor. Doing so may cause you to fall or slip, resulting in serious injury. It may also cause water to get inside the monitor resulting in damage to internal components.

Caution

	Do not stand on the display screen or control panel.
	Do not insert your fingers into gaps or holes. Doing so may result in injury.
	Do not use rechargeable batteries. Do not use old batteries together with new batteries, or a mix of different types of batteries at the same time. Doing so may cause battery fluid to leak or the batteries to become excessively hot and rupture, resulting in damage to the monitor or injury.
	Never allow an infant or small child to carry the monitor. The child may drop the monitor, resulting in injury.
	Store out of reach of small children. Small children may become caught up in the cables, resulting in injury.
	This monitor is intended for home use only. This monitor is not intended for professional use including hospitals, medical or fitness facilities. It is not equipped with the components required for heavy usage. Using the monitor in a professional location will invalidate the warranty.

	Place the monitor on a hard, flat surface where there is minimal vibration to ensure safe and accurate measurement.
	Do not use or store this monitor in the following locations. · Locations that are exposed to direct sunlight, near to heaters, or exposed directly to air from air conditioning units, etc. · Locations where there are severe temperature changes or a large amount of vibration. · Locations with high levels of humidity or moisture Doing so may damage the monitor or cause it to malfunction.
	Do not step on the monitor when your body is wet (immediately after a shower or bath, etc.) Be sure to thoroughly dry your body and feet before using the monitor. Failure to do so may cause you to slip or fall, resulting in injury.
	Do not use boiling water, benzene, thinner or alcohol, etc. to clean the monitor. If the monitor becomes dirty, soak a soft cloth in water or neutral household detergent, wring it out thoroughly and wipe the monitor clean, then use a dry cloth to wipe the monitor dry.
	Do not submerge this monitor in water. Do not attempt to disassemble the monitor. Do not modify this equipment in any way. Do not wash the monitor in water. Doing so may damage the monitor or cause it to malfunction.
	Be sure to insert the batteries with the polarities (+, -) in the correct position. If the batteries are incorrectly inserted, battery fluid may leak and damage the floor. (If you do not intend to use this monitor for a long period of time, it is advisable to remove the batteries before storage.)
	Do not use the monitor near other products such as microwave ovens that emit electromagnetic waves. Doing so may cause the monitor to malfunction, preventing accurate measurements.
	Another person should assist persons with disabilities who may not be able to take a measurement alone.

1. Specifications

Capacity: 150kg/ 330lb/ 23st 8lb
Graduation: 0.1kg/ 0.2lb/ 1lb
Battery Type: (Fig. 2) CR2032, (Fig. 3) AAA x 2

2. Parts (Fig. 1)

- Body
- Display
- Battery (Bottom)

3. Before Use

Select your preferred weight mode by using the Weight Mode Switch.

4. Weighing

- Gently push down the middle of the platform with your foot and release the pressure to activate the scale.
 - When "0.0" appears in the display, step onto the platform, and stand still.
 - Within seconds your weight will lock and then flash.
 - The scale will turn off automatically after 30 seconds.
- Note: The LCD will display "OL", if the weight is over 150kg/ 330lb/ 23st 8lb, and will turn off after 30 seconds. If irregular display appears while measuring, reset the scale by removing and reinserting the batteries.

5. Replacing the Batteries

When the low battery sign (Lo) appears on the LCD, remove the current batteries and replace them with new set as indicated in Fig 2 and 3.

- The product design and specifications may be changed at any time without prior notice.

Disposal

This product is an electronic device. Please dispose of this product appropriately, not as general household waste. Be sure to follow the regulations in your area when disposing of this product.

Not allowed to mix batteries with consumer wastes!
As consumer you are legally bound to return used or discharged batteries. You can deposit your old batteries at the public collecting points in your town, or wherever the corresponding batteries are sold and specifically marked collecting boxes have been set up. In case of scrapping the apparatus, the batteries should be removed from it and deposited at the collecting points as well.

This device meets the EMC Directive 2014/30/EU and the RoHS Directive 2011/65/EU.

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TANITA Corporation
TANITA Health Equipment H.K. Ltd.
TANITA Europe B.V.
TANITA (Shanghai) Trading Co., Ltd.
TANITA Corporation of America, Inc.
Arlington Heights, Illinois 60005 U.S.A.